
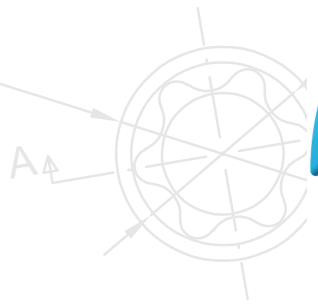


DESS

> DENTAL SMART SOLUTIONS

CLINICAL SCREWS RECOMMENDED TORQUE

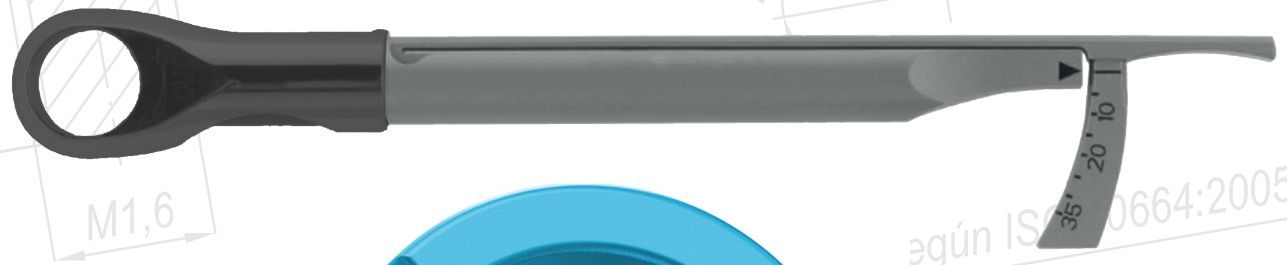

según ISO 664:2005

ACTIVE HEX

FITS
NOBELACTIVE® &
NOBELREPLACE® CC



PLATFORM SIZE	DESS REF. N°	DESS RECOMMENDED TORQUE (Ncm)	ORIGINAL EQUIVALENT REF. N°
3.0	19.068	15	Nobel Biocare® 36805
3.0	19.268	15	Nobel Biocare® 37890
3.0	19.668	15	
NP	19.027	35	Nobel Biocare® 31168
NP	19.227	35	Nobel Biocare® 29282
NP	19.101	35	
NP	19.327	35	
NP	19.627	35	
NP AURUMBase®	19.441	35	
RP	19.030	35	Nobel Biocare® 28815
RP	19.230	35	Nobel Biocare® 36918
RP	19.328	35	
RP	19.630	35	
RP AURUMBase®	19.442	35	
On1®	19.021	35	Nobel Biocare® 38751
On1®	19.221	35	Nobel Biocare® 38750
On1®	19.621	35	
NP on implant for Multi-unit®	19.507	35	
RP on implant for Multi-unit®	19.508	35	

TRI-LOBE

FITS
NOBEL REPLACE
SELECT™



NP	19.004	35	Nobel Biocare® 31170
NP	19.204	35	Nobel Biocare® 36818
NP	19.304	35	
NP	19.604	35	
NP AURUMBase®	19.404	35	
RP/WP/6.0	19.005	35	Nobel Biocare® 29293
RP/WP/6.0	19.205	35	Nobel Biocare® 29475
RP/WP/6.0	19.305	35	
RP/WP/6.0	19.605	35	
RP/WP AURUMBase®	19.405	35	
NP on implant for Multi-unit®	19.504	35	
RP on implant for Multi-unit®	19.505	35	

EXT HEX UNIVERSAL

FITS
NOBELBRÅNEMARK*



PLATFORM SIZE	DESS REF. N°	DESS RECOMMENDED TORQUE (Ncm)	ORIGINAL EQUIVALENT REF. N°
NP	19.027	35	Nobel Biocare® 31168
NP	19.227	35	Nobel Biocare® 29282
NP	19.327	35	
NP	19.101	35	
NP	19.627	35	
NP AURUMBase®	19.401	35	
RP	19.028	35	Nobel Biocare® 29290
RP	19.228	35	Nobel Biocare® 29283
RP	19.328	35	
RP	19.628	35	
RP AURUMBase®	19.402	35	
WP	19.029	35	Nobel Biocare® 31169
WP	19.229	35	Nobel Biocare® 29284
WP	19.329	35	
WP	19.629	35	
NP on implant for Multi-unit®	19.501	35	
RP on implant for Multi-unit®	19.502	35	

EXT HEX USA

FITS
3i OSSEOTITE*



NP/RP/WP	19.034	20	Biomet 3i® UNIHT
NP/RP/WP	19.634	20	
NP/RP/WP AURUMBase®	19.402	20	
RP on implant for Multi-unit®	19.512	20	
Low Profile® Abutment ø4.8	19.106	10	Biomet 3i® LPCTSH

INT HEX CLIK

FITS
3i CERTAIN*



NP/RP/WP	19.011	20	Biomet 3i® IUNIHT
NP/RP/WP	19.311	20	
NP/RP/WP	19.611	20	
NP/RP/WP AURUMBase®	19.438	20	
Low Profile® Abutment ø4.8	19.106	10	Biomet 3i® LPCTSH

OCTAGON

FITS
STRAUMANN® SOFT
TISSUE LEVEL &
synOCTA*



NNC	19.070	35	Straumann® AG 048.314
NNC	19.670	35	
RN/WN on implant	19.026	35	Straumann® AG 048.356
RN/WN on implant	19.102	35	Straumann® AG 049.181
RN/WN on implant	19.328	35	
RN/WN on implant	19.626	35	
RN/WN AURUMBase®	19.446	35	
RN/WN on synOcta® abutment	19.009	15	Straumann® AG 048.350
RN/WN on synOcta® abutment	19.103	15	
RN/WN on synOcta® abutment	19.609	15	

CONICAL BL

FITS
STRAUMANN®
BONE LEVEL



PLATFORM SIZE	DESS REF. N°	DESS RECOMMENDED TORQUE (Ncm)	ORIGINAL EQUIVALENT REF. N°
NC on implant	19.032	35	Straumann® AG 025.2908
NC on implant	19.632	35	
RC on implant	19.033	35	Straumann® AG 025.4908
RC on implant	19.633	35	
NC/RC on implant	19.037	35	Straumann® AG 025.4900
NC/RC on implant	19.104	35	Straumann® AG 025.4906
NC/RC on implant	19.311	20	
NC/RC AURUMBase®	19.443	35	
NC/RC on implant for Multi-unit®	19.544	35	
NC/RC on Multi-base® abutment	19.041	15	Straumann® AG 025.0900-04
NC/RC on Multi-base® abutment	19.105	15	
NC/RC on SRA® abutment	19.098	15	Straumann® AG 023.4763

INT HEX USA

FITS
ZIMMER
SCREW-VENT®



NP/RP/WP	19.012	30	Zimmer® URS2
NP/RP/WP	19.012L	30	
NP/RP/WP	19.042	30	Zimmer® MHLAS
NP/RP/WP	19.642	30	
NP/RP/WP	19.312	30	
NP/RP/WP AURUMBase®	19.417	30	
NP (MIS®)	19.327	35	
NP on implant for Multi-unit®	19519	30	
RP on Multi-unit® abutment ø4.8	19.110	15	
RP on Multi-unit® abutment ø4.8	19.300	15	

CONIC EVO

FITS
ASTRA TECH
IMPLANT
SYSTEM™ EV



3.6 on implant	19.059	25	
3.6 on implant	19.659	25	
3.6 AURUMBase®	19.441	25	
4.2 on implant	19.060	25	
4.2 on implant	19.660	25	Astra Tech® 25.205
4.2 AURUMBase®	19.460	25	
4.8 on implant	19.061	25	
4.8 on implant	19.661	25	
4.8 AURUMBase®	19.461	25	
UniAbutment® EV	19.058	15	
UniAbutment® EV	19.658	15	

INT HEX CONIC

FITS
ASTRA TECH
OSSEOSPEED™



NP on implant	19.015	15	Dentsply® 24189
NP on implant	19.615	15	
RP on implant	19.016	20	Dentsply® 24449
RP on implant	19.616	20	
RP AURUMBase®	19.441	20	
WP on implant	19.017	25	Dentsply® 24209
WP on implant	19.617	25	
WP AURUMBase®	19.425	25	
UniAbutment® 20°/45°	19.018	15	Dentsply® 22435
UniAbutment® 20°/45°	19.618	15	

INT HEX FD

FITS
DENTSPLY
FRIADENT XIVE*



PLATFORM SIZE	DESS REF. N°	DESS RECOMMENDED TORQUE (Ncm)	ORIGINAL EQUIVALENT REF. N°
NP/RP/WP	19.024	24	Dentsply® 454305
NP/RP/WP	19.624	24	
NP/RP/WP AURUMBase®	19.438	24	

INT ANK

FITS
DENTSPLY
ANKYLOS® C/X



All platforms	19.048	15	
All platforms	19.050	15	
All platforms	19.648	15	

CONIC OSS

FITS
OSSTEM® TS



Mini on implant	19.072	20	
Mini on implant	19.672	20	
Mini AURUMBase®	19.444	20	
Standard on implant	19.073	30	Osstem® GSABSST
Standard on implant	19.673	30	
Standard AURUMBase®	19.461	30	

INT CAM

FITS
CAMLOG®



3,3 / 3,8 / 4,3 on implant	19.044	20	
3,3 / 3,8 / 4,3 on implant	19.644	20	Camlog® J4005. 1601
3,8 / 4,3 AURUMBase®	19.444	20	
5.0 on implant	19.045	20	Camlog® J4005. 2001

CONIC ANYR

FITS
MEGAGEN AnyRidge®



All platforms	19.057	35	
All platforms	19.657	35	Megagen® AANMSF
All platforms AURUMBase®	19.460	35	

All DESS products are manufactured according to ISO 9001:2015 and ISO 13485:2016 with CE marking for all products Class IIb

All DESS® AngleBase® screws and screwdrivers are based on the standard ISO 10664

3i® and 3i Certain®, Miniplant®, MicroMiniplant™ and Osseotite® are registered trademarks of 3i Implant Innovations, USA

ASTRA TECH™, OsseoSpeed™, ANKYLOS® C/X, XIVE®, FRIADENT®, ASTRATECH implant system™ EV and UniAbutment®

are registered trademarks of DENTSPLY SIRONA Implants

Brånemark System®, NobelReplace® and Replace Select™, NobelActive™, NobelReplace® CC, NobelSpeedy™ and Multi-unit® are

registered trademarks of Nobel Biocare AB, Sweden

ITI®, Straumann® and synOcta® are registered trademarks of Straumann Holding AG, Switzerland

MIS® is a registered trademark of MIS Implant Technologies Ltd.

Zimmer Screw-Vent® and SwissPlus® are registered trademarks of Zimmer Dental INC, USA

Locator® is a registered trademark of ZEST IP Holdings

Camlog® are registered trademarks of CAMLOG Biotechnologies AG

Megagen® is a registered trademark of MEGAGEN IMPLANT CO. LTD

TORX® is a trademark of ACUMENT INTELLECTUAL PROPERTIES, LLC

Osstem® is a registered trademark of OSSTEM IMPLANT CO. LTD

DESS products can be used only by licensed dentists, physicians and labs.

Please refer ever to the instructions of use available on the website

www.dess-abutments.com

or supplied with the product.

Some products may not be available in your country. Some of the products are cleared for US and Canada.

Please contact your local distributor for more details.